

WHAT DO I PUT THIS IN?

TWO PATIENT IDENTIFIERS
must be on **EACH SPECIMEN**
(e.g. **NAME** and **DOB**)

COLLECTION VIALS



ThinPrep

- Anal pap test (dacron swab)
- GYN pap test
- HPV
- Chlamydia/Gonorrhea
- Trichomonas
- Herpes (HSV)
- Molecular panel testing for BV, Candida Species, and STIs
- Specimen viable for 6 weeks



CryoRich® Red Fluid Vial

- EMB specimens
- ECC (endocervical curettage)
- Soft ECC® specimens
- Non-GYN cytology



Gray Urine Vial

- Urine culture & sensitivity
- Specimen expires in 48 hours



Orange Top Urine Cup Aptima Urine Vial

- Urine Cytology (Orange Top Urine Cup only)
- Orange Top expires in 12 hours, refrigerated
- Molecular panel testing for STIs: Chlamydia/Gonorrhea, Trichomonas, Ureaplasma, Mycoplasma
- Yellow vial expires in 7 days



Formalin Container

- Tissue biopsy specimens
- LEEP specimens
- Soft Bx® specimens
- EMB specimens



BBL Culture Swab

- Group A Strep
- Group B Strep
- Group B Strep Culture & Sensitivity testing
- Yeast culture & Sensitivity testing
- Specimen expires in 48 hours



Aptima Swab

- Chlamydia/Gonorrhea/ Trichomonas
- Herpes (HSV)
- Molecular panel testing for BV, Candida Species, and STIs
- Specimen viable for 8 weeks



Affirm Swab

- Vaginosis panel (Candida, Trichomonas, Gardnerella)
- Specimen expires in 3 days



Swab in Amies Agar Gel (Blue Top)

- Aerobic and anaerobic bacterial culture
- Specimen expires in 72 hours at ambient/room temperature



Red/Yellow Urinalysis Tubes

- Used for Urinalysis
- Specimen expires in 48 hours



Hologic Fetal Fibronectin Swab


- Fetal Fibronectin test
- Specimen expires in 8 hours (ambient) or 3 days (refrigerated)
- Please call Laboratory if STAT or RUSH pickup is required

BLOOD COLLECTION TUBES



Red Top*

- Acid Phosphatase
- Specimen must be frozen
- Please call Laboratory for FROZEN pickup

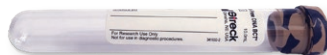

DNA Streck Tube

- Panorama™ Non-Invasive Prenatal Test
- Specimen expires in 5 days



Pink Top

- Blood type (ABO, Rh testing)
- Antibody screen



Green Top

- Sodium Heparin
- Special testing (chromosomal analysis)



Light Blue Top*

- Coagulation studies
- Specimen must be frozen
- Please call Laboratory for FROZEN pickup



Tiger Top or SST

- Chemistry
- Infectious Diseases
- Hormone Analysis
- Immunology/Rheumatology
- Oncology



Royal Blue (Lavender Strip)

- K2 EDTA
- Trace element analysis (Heavy metals, Lead, Arsenic, Cadmium and others)

Royal Blue (Red Strip)

- No Additive
- Trace element analysis (Aluminum, Chromium, Iodine, and others)



4 ml Lavender Top

- CBC
- Hemoglobin A1C

6 mL Lavender Top

- Genetic Carrier Screening
- Hereditary Cancer Screening

*Saliva vial available upon request.



PathAdvantage

To order more supplies Call 214.219.5880
or email pacouriers@pathadvantage.com

*Call 214-219-5880 for FROZEN pickup